

**Ronjair Equipment**  
***Sterilization Instructions***

After each operation:

1. Detach hose from power handle and apply two drops of AIR DRIVER or CRANIOTOME Oil to stand pipe.
2. Reattach hose to power handle and actuate trigger four or five times.
3. Clean and disassemble attachments, blades and hose.
4. Do not immerse handle in liquids.
5. Do not clean power handle in ultrasonic cleaner.
6. Blades can be cleaned in ultrasonic cleaner.
7. When not in use, store in an autoclave case.
8. Entire unit is autoclavable or gas sterilizable.

