

# Care and Maintenance Instructions

## Synthes Nail Cutter (359.217)



The Cutter [359.217] has removable parts. Prior to cleaning, disassemble the cutter by unscrewing the stop nut and removing the cutting bolt from the cutting sleeve.

Steel brushes must not be used to clean the instruments. Standard proprietary detergents and disinfectants can be used in accordance with the manufacturers' recommendations.

Prior to autoclaving, instruments should be checked for cleanliness. Lubricate the cutting bolt with autoclavable oil. The cutting bolt must be adequately lubricated to ensure smooth cutting. To reassemble the cutter, insert the cutting bolt into the cutting sleeve. Screw the stop nut onto the assembly.

